

# ***Vacuum mats for Surgical Positioning***



# Vacuum mats for patient positioning during surgery

- Conforms to each patients body and body part
- Removes pressure points
- Quick, safe and easy positioning

## Great benefits:

- Effective prevention of decubitus and paresis
- Optimum access for the surgeon to the operating site
- Excellent patient comfort
- Improved efficiency in the operating room
- Making Inter-clinic transportation in defined surgical position possible
- Can be used as reusable alternative to plaster beds

resulting in **GREAT COST SAVINGS**



## Indications to use the MedVac vacuum positioning system during surgery:

- Side position
- Half-side position
- Back position
- Sitting / half- sitting position
- Coiled position
- Lithotomy position
- Laparoscopic positions


## How to operate the MedVac system:

- Make sure the inner granulate material is evenly distributed and the vacuum mat is flat
- Vacuum the mat slightly
- Place the patient on the mat
- Open the valve to let a little air into the mat making it soft
- Put the patient in the required position
- Vacuum the mat until it is firm

## Features of all MedVac products:

### Special biocompatible high quality polyurethane film:

- Durable, colourfast material guarantees long life
- Soft as silk on the patient, still extremely tough and puncture resistant
- Clean design without places that can trap fluids and dirt makes it very easy to clean and disinfect.
- Biocompatible, free from PVC and latex

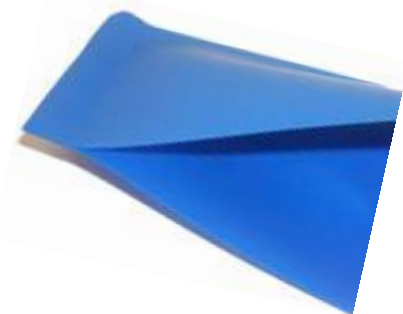

### Automatic vacuum valves:

- The vacuum valves close automatically when suction stops
- No risk of losing vacuum if slipping with the vacuum pump



Removable valve



Non-removable valve

### Labels on every product:


- Individual label with ID-number
- Label with instruction how to operate the valve (optional)
- Label with distributor logo and information (optional)
- Label with customer logo and information (optional)



Compatible with X-ray, CT and MRI. CE marked


# MedVac Surgical Positioning Vacuum Mattresses :

## Surgical mattresses, body size:

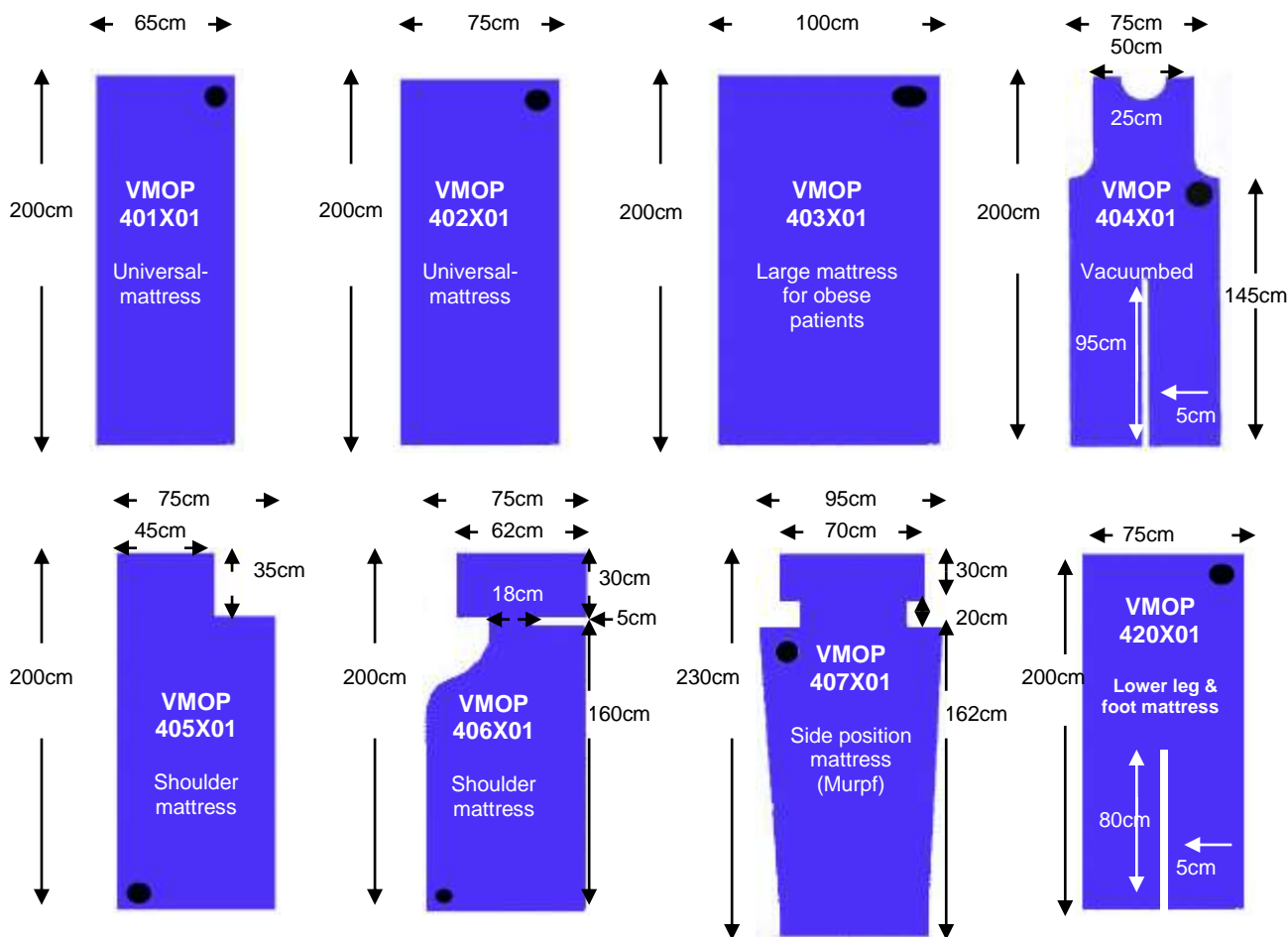


## Short surgical mattresses:


### Short mattresses with cut out:

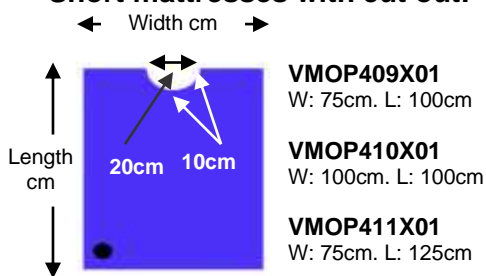

### Short mattresses:

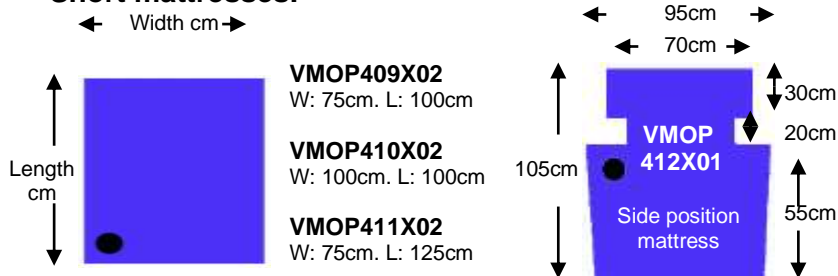

## Surgical cushions:

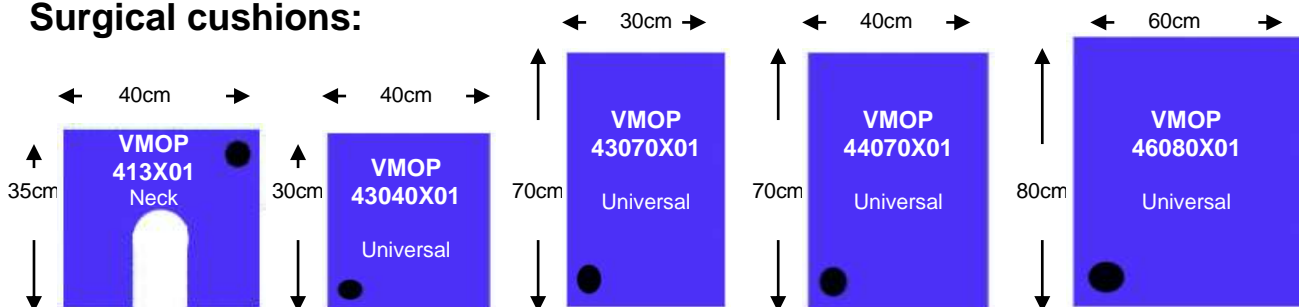

The logo for Kohlbrat & Bunz features the company name in a bold, black, sans-serif font. The ampersand is white and set within a red oval. The entire logo is enclosed in a large, thin red oval that curves around the text.

**KOHLBRAT & BUNZ**

*Medical Products  
Made in Austria*

Quality management in accordance with  
EN ISO 9001 - EN ISO 13485

certified by



*made by*

**KOHLBRAT & BUNZ GMBH**  
Loretostrasse 4-8  
5550 Radstadt - Austria  
T: +43 6452 7193 0  
F: +43 6452 7193 51  
E: [office@kohlbrat-bunz.com](mailto:office@kohlbrat-bunz.com)  
[www.kohlbrat-bunz.com](http://www.kohlbrat-bunz.com)