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
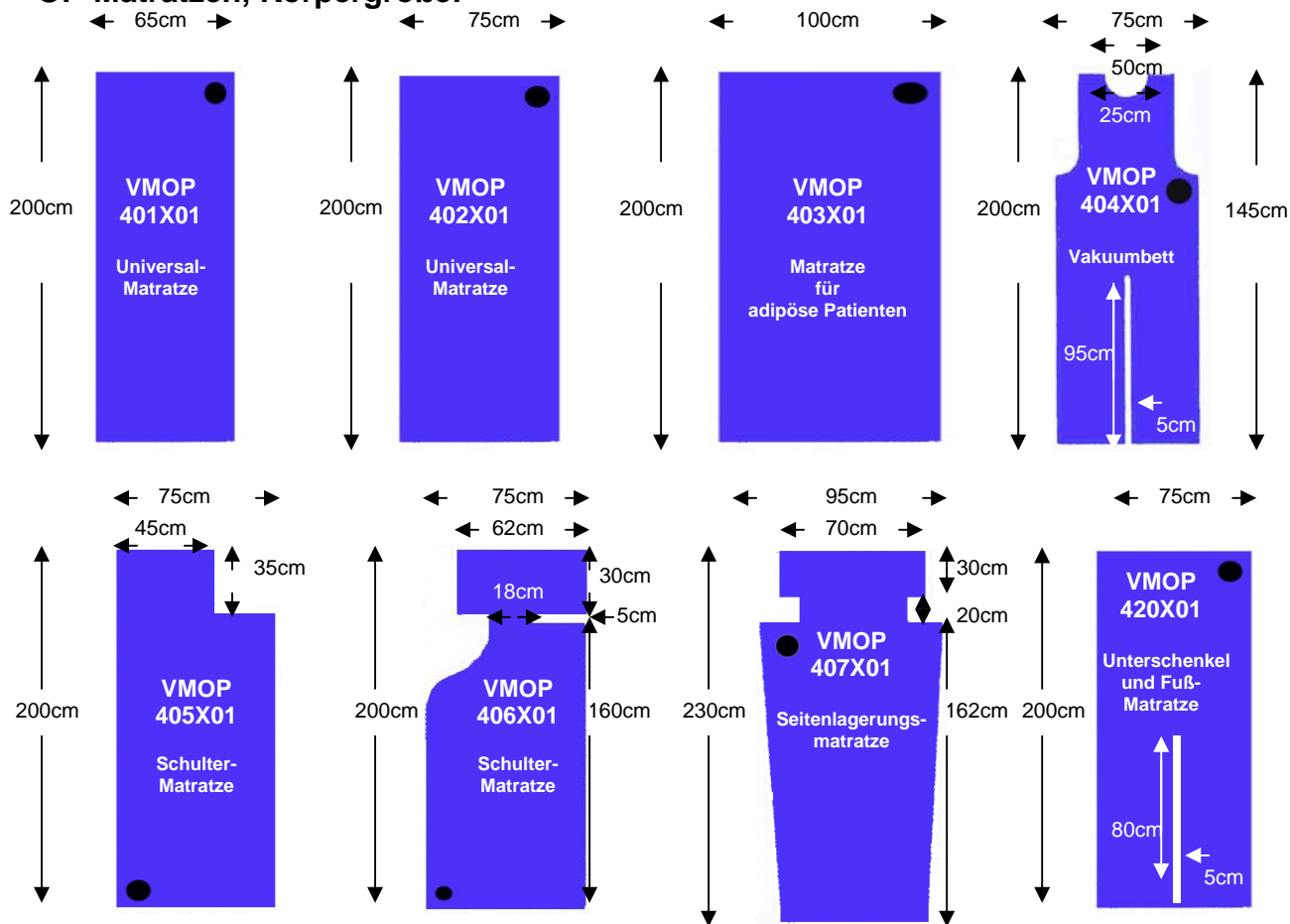
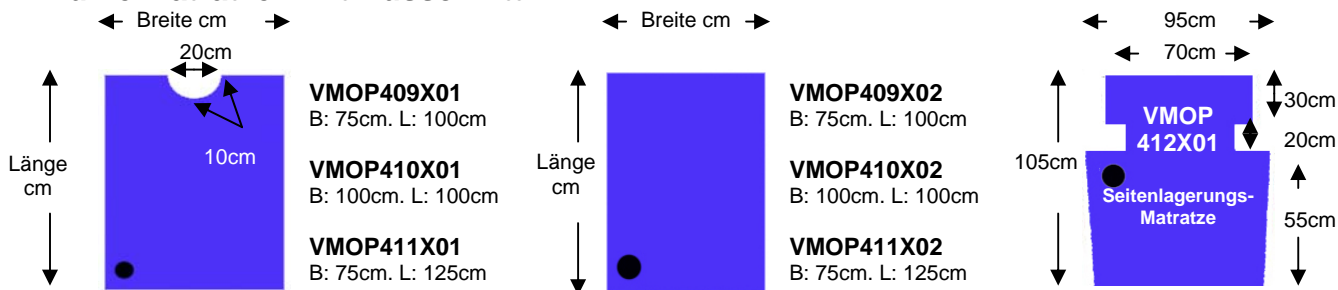
Kurze Matratzen:

OP-Kissen:

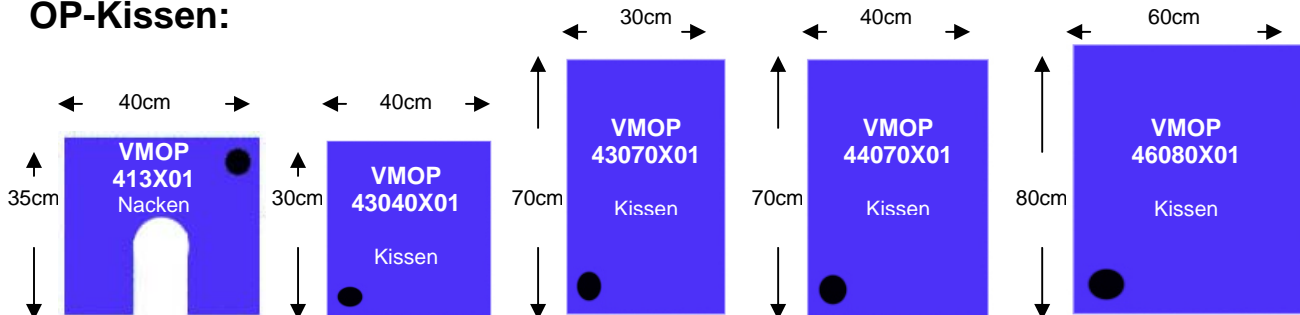The logo for Kohlbrat & Bunz features the company name in a bold, black, sans-serif font. The ampersand is enclosed in a red oval. The entire logo is framed by a large, red, curved line that forms an open oval shape.

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Vacuum mats for Surgical Bedding

Vacuum mats for patient positioning during surgery

- Conforms to each patients body and body part
- Removes pressure points
- Quick, safe and easy positioning

Great benefits:

- Effective prevention of decubitus and paresis
- Optimum access for the surgeon to the operating site
- Excellent patient comfort
- Improved efficiency in the operating room
- Making Inter-clinic transportation in defined surgical position possible
- Can be used as reusable alternative to plaster beds

resulting in **GREAT COST SAVINGS**

Indications to use the MedVac vacuum positioning system during surgery:

- Side position
- Half-side position
- Back position
- Sitting / half- sitting position
- Coiled position
- Lithotomy position
- Laparoscopic positions

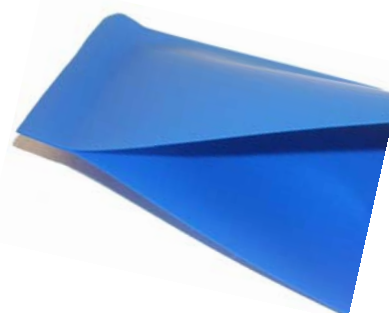
How to operate the MedVac system:

- Make sure the inner granulate material is evenly distributed and the vacuum mat is flat
- Vacuum the mat slightly
- Place the patient on the mat
- Open the valve to let a little air into the mat making it soft
- Put the patient in the required position
- Vacuum the mat until it is firm

Features of all MedVac products:

Special biocompatible high quality polyurethane film:

- Durable, colourfast material guarantees long life
- Soft as silk on the patient, still extremely tough and puncture resistant
- Clean design without places that can trap fluids and dirt makes it very easy to clean and disinfect.
- Biocompatible, free from PVC and latex

Automatic vacuum valves:

- The vacuum valves close automatically when suction stops
- No risk of losing vacuum if slipping with the vacuum pump

Labels on every product:

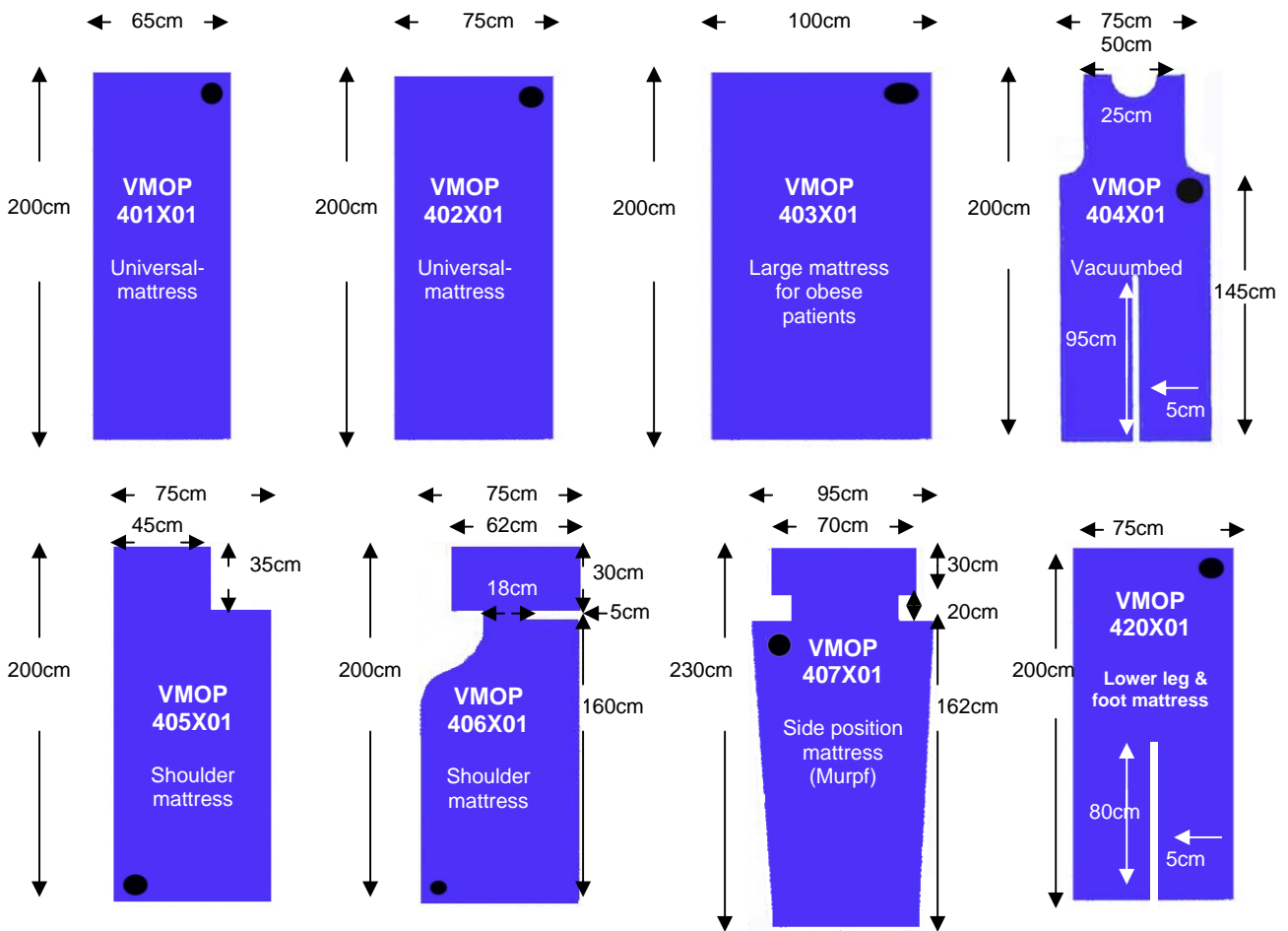
- Individual label with ID-number
- Label with instruction how to operate the valve (optional)
- Label with distributor logo and information (optional)
- Label with customer logo and information (optional)

Compatible with X-ray, CT and MRI. CE marked


MedVac Surgical Vacuum Mattresses :

Surgical mattresses, body size:


Short surgical mattresses:

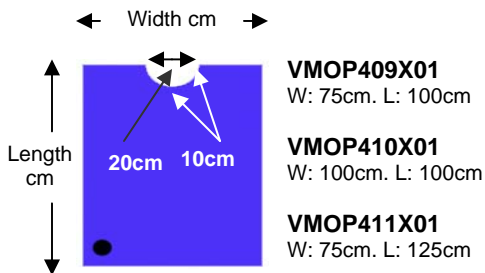
Short mattresses with cut out:

Short mattresses:

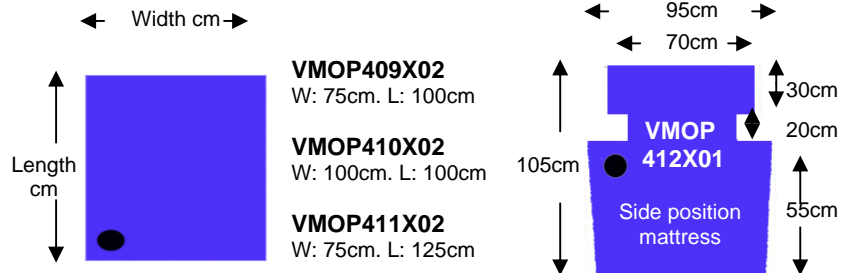Surgical cushions:

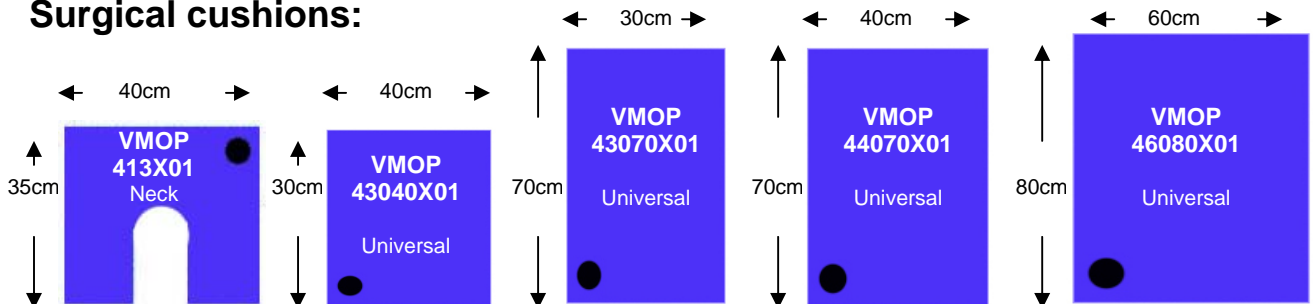The logo for Kohlbrat & Bunz features the company name in a bold, black, sans-serif font. The ampersand is enclosed within a red oval. The entire logo is framed by a large, red, curved swoosh that starts above the text and ends below it, creating a dynamic, open-ended shape.

KOHLBRAT & BUNZ

*Medical Products
Made in Austria*

**Quality Management according to
EN ISO 9001 & EN ISO 13485
certified by TÜV Austria**

made by

**KOHLBRAT & BUNZ GMBH
Loretostrasse 4-8
5550 Radstadt - Austria
T: +43-(0)6452-7193-0
F: +43-(0)6452-7193-51
E: office@kohlbrat-bunz.com
www.kohlbrat-bunz.com**